PRODUCT USE:

The MILTEX BIOPBLADE® is utilized for removal of lesions, either elevated (shave biopsy) or flat (saucerization).

INDICATIONS FOR USE:

The MILTEX BIOPBLADE® is indicated for use by a licensed medical professional to obtain a biopsy of any soft tissue. This is a disposable product and is not sterilizable.

WARNINGS/PRECAUTIONS:

- Use caution when handling and disposing of this product as the blade is sharp.
- Do not use this product if the individual bag is broken as sterilization cannot be guaranteed.
- The MILTEX BIOPBLADE® is a disposable product intended for single use applications.
- Do not place on or near any products containing chlorides as rusting may occur.

CONTRAINDICATIONS/LIMITATIONS: None known.

ADVERSE REACTIONS OR SIDE EFFECTS: None known.

HOW SUPPLIED: Each MILTEX BIOPBLADE® is individually sealed in a pouch to ensure sterility. These products are sold in boxes of 50.

STEP-BY-STEP INSTRUCTIONS:

1. After the site is anesthetized, remove MILTEX BIOPBLADE® from its packaging.
2. Hold the MILTEX BIOPBLADE® “bowed” between the thumb and fingers. (see fig. 1)
3. NOTE the “UP” on the MILTEX BIOPBLADE®. This should face up. (see fig. 2)
4. Remove the lesion at or just below the surface epithelium.
5. Dispose of the MILTEX BIOPBLADE® as you would dispose of a used needle.

NOT RECOMMENDED/INAPPROPRIATE USES: None known.

OTHER IMPORTANT INFORMATION: None known.

SYMBOLS USED ON LABELING

See Instructions for Use

Consult Instructions for Use

Use by: Year-Month

Single Patient Use

Batch Code / Lot Number

Catalog Number

To be used by or on the order of a licensed medical professional.

Authorized Representative in the European Community


STERILE – STERILIZED BY IRRADIATION
Sterility guaranteed unless package opened or damaged. Do not re-sterilize.

Product complies with requirements of directive 93/42/EEC for medical devices

fig. 1

fig. 2

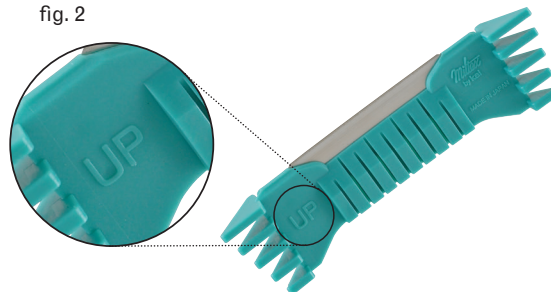Manufactured for:
Integra York PA, Inc.
589 Davies Drive
York, PA 17402
toll free phone: 866-854-8300
phone: 717-840-9335
fax: 717-840-9347
www.miltex.com

Integra LifeSciences Services (France) SAS Immeuble Sequoia 2
97 allée Alexandre Borodine
Parc Technologique de la Porte des Alpes
69800 Saint Priest–France
Telephone: 33 (0) 4 37 47 59 10
Fax: 33 (0) 4 37 47 59 29

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