

## INSTRUCTIONS FOR USE - PADGETT DISPOSABLE (STERILE) BIPOLAR INSTRUMENT CABLE

### CAUTION

Please read all of the following information and instructions prior to use. Federal (USA) law restricts this device to sale by or on the order of a physician.

### HOW SUPPLIED

This product is disposable, supplied sterile and intended for single use only. Prior to the expiration date, contents are sterile if the package has not been opened or damaged. Do not re-sterilize.

### INDICATIONS FOR USE

Disposable bipolar instrument cords are designed to connect an electrosurgical device, such as Bipolar Forceps, to a compatible electrosurgical generator. In particular, the Padgett disposable sterile bipolar instrument cables are intended for use with bipolar forceps during general surgical procedures. We recommend that you review the Instructions for Use on PADGETT BIPOLAR FORCEPS (Ref. No. PAD-IFU-BIPLR). These instructions are readily available and downloadable from our website at [www.miltex.com](http://www.miltex.com).

### CONTRAINDICATIONS

Any use of this instrument cable for tasks other than for which is it indicated, can lead to premature wear, result in a damage or can cause hazards to patients and users.

**CAUTION & WARNING – CONNECT BIPOLAR ACCESSORIES TO THE BIPOLAR RECEPTACLE ONLY AND MONOPOLAR ACCESSORIES TO THE MONOPOLAR RECEPTACLE ONLY. DO NOT ATTEMPT TO CONNECT OR INTERCHANGE BIPOLAR AND MONOPOLAR ACCESSORIES AND RECEPTACLE CONNECTIONS. IMPROPER CONNECTION OF ACCESSORIES MAY RESULT IN INADVERTENT ACCESSORY ACTIVATION OR OTHER POTENTIALLY HAZARDOUS CONDITIONS.**

### ADVERSE REACTIONS or SIDE-EFFECTS

None known.

### SETUP and USE

Attach the sterile Bipolar cord to the sterile Bipolar instrument, ensuring that the contact pins are fully seated in the cable receptacles. Connect the cord to the generator only while it is in the "OFF" or "STANDBY" mode. Failure to do so may result in injury or electrical shock to the patient or operating room personnel.

At the lowest power setting, test the instrument connections by pressing the generator's activating switch. If the generator fails to activate, check the line connection with the cable. Always review and follow generator manufacturer operating and use instructions.

### RETURNED GOODS POLICY

Products must be returned with proof of purchase and in unopened packages with manufacturer's seals intact to be accepted for replacement or credit unless returned due to a complaint or product defect. Determination of a product defect will be by Miltex. Products will not be accepted for replacement if they have been in the possession of the customer for more than 120 days.

### PRODUCT INFORMATION DISCLOSURE

Miltex, Integra and manufacturer exclude all warranties, whether expressed or implied, including but not limited to, any implied warranties or merchantability or fitness for a particular purpose. Neither Miltex, Integra nor manufacturer shall be liable for any incidental or consequential loss, damage, or expense, directly or indirectly arising from use of this product. Neither Miltex, Integra nor Manufacturer assume nor authorize any person to assume for them any other or additional liability of responsibility in connection with these products.

#### SYMBOLS USED ON LABELING



See Instructions for Use



Consult Instructions for Use



Use by: Year-Month



Single Patient Use



Batch Code / Lot Number



Catalog Number

Manufactured for:  
Miltex, Inc.  
589 Davies Drive  
York, PA 17402  
toll free phone: 866-854-8300  
phone: 717-840-9335  
fax: 717-840-9347  
[www.miltex.com](http://www.miltex.com)

**Rx ONLY**

CAUTION: USA Federal Law restricts this device to sale by or on the order of a physician only.

Padgett is a registered trademark of Integra LifeSciences Corporation. The Integra wave logo is a trademark of Integra LifeSciences Corporation. ©2006 Integra LifeSciences Corporation. All Rights Reserved.

**STERILE R**

STERILE – STERILIZED BY IRRADIATION  
Sterility guaranteed unless package opened or damaged. Do not re-sterilize.

IFU-PM9450D

0907/0/A/1