# Integra® Miltex®

Care and Handling of  
Dental and Endodontic Devices  
Directions for use / English

## Rx ONLY

### Directions For Use

Rx ONLY CAUTION: Federal (USA) law restricts this device to sale by or on the order of a dentist or physician.

### Contraindications

None Known.

### Precautions

- Instruments/devices should be inspected for proper function prior to use.
- Sterilize all hinged instruments with hinges in open position.
- For GripLite® Instruments and hand files, do not use dry heat sterilization or expose to phenols or iodophors.

### Cleaning Instructions

#### Automated Cleaning

1. Place instrument/device into the ultrasonic basket, sterilization tray or cassette.
2. Place the ultrasonic basket of instruments/devices, sterilization tray or cassette into the ultrasonic cleaner or washer.
3. Follow the manufacturer's instructions for the appropriate cleaning cycle for the instruments/devices being cleaned.
4. Thoroughly rinse the instrument/device, sterilization tray or cassette under warm tap water.
5. Inspect the instrument/device for any remaining debris and repeat steps 1 through 4 if debris remains on the instrument/device.
6. Allow the instrument/device to completely dry prior to proceeding to the sterilization process.

#### Manual Cleaning

1. Remove any visible debris or foreign matter from the surface of the instrument/device by scrubbing the instrument/device with a cleaning brush and a pH 7 detergent such as Integra® Miltex® Surgical Instrument Cleaner.
2. Thoroughly rinse the instrument/device under warm tap water.
3. Inspect the instrument/device for any remaining debris and repeat steps 1 and 2 if debris remains on the instrument/device.
4. Allow the instrument/device to completely dry prior to moving to the sterilization process.

### Sterilization Instructions

- Sterilize instrument/device according to an approved National/International Standard. Integra® Miltex® instruments/products have been tested with no adverse effects under the following sterilization parameters in a standard table top style sterilizer:
  - Temperature: 250°F (121°C)
  - Exposure Time: 30 minutes
- Do not sterilize size 140 files with stops in place.

### Symbols Used on Labeling

Manufacturer 1

European Authorized Representative

Catalog Number

Lot Number

Caution: See Warnings and Precautions

Consult Instructions for Use

Complies with MDD 93/42/EEC

Complies with MDD 93/42/EEC

Federal (USA) law restricts this device to sale by or on the order of a physician or practitioner

Integra York PA, Inc.  
589 Davies Dr.  
York, PA 17402 USA  
866-854-9300  
717-840-9335  
www.integralife.com/integra-miltex

Integra LifeSciences Services (France) SAS Immeuble Sequoia 2  
97 allée Alexandre Borodine  
Parc Technologique de la Porte des Alpes  
69800 Saint Priest-France  
Telephone: 33 (0) 4 37 47 59 10  
Fax: 33 (0) 4 37 47 59 29

<sup>1</sup> Company responsible for a device marketed under its own name regardless of whether "manufactured for" or "manufactured by" the company.

Integra, the Integra logo, Miltex, and GripLite are registered trademarks of Integra LifeSciences Corporation or its subsidiaries in the United States and/or other countries.  
©2012 Integra LifeSciences Corporation. All Rights Reserved. LPDFU Rev. D 07/13

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