MILTEX® PORTABLE LED LOUPES LIGHT SOURCE DIRECTIONS FOR USE

Includes warranty and operating instructions for the
Miltex® LED Loupes Light Source

MILTEX® PORTABLE LED LOUPES LIGHT SOURCE

DIRECTIONS FOR USE

Indications: The LED Loupes Light Source is intended for use **ONLY** in conjunction with magnification loupes and should not be used without them. The light source is used to illuminate the magnified field only.

Rx ONLY

Rx Only: Caution: Federal (US) law restricts this device to sale by or on the order of a physician.

Contents:

1. LED lamp head and cable with loupes mount
2. Power pack with one lithium-ion rechargeable battery
3. Battery charger and AC wall adapter

1. Charge battery

Plug the power adapter into a power outlet and connect to charger unit.

- a. A single red light will indicate power to the unit. If the red light does not appear, remove the battery from the charger, unplug the power adapter from the charger unit and wait 5 minutes before reconnecting. If the red light still does not appear, call Miltex Customer Support.
- b. Slide the battery into the Charger Unit. Be sure the battery terminals are securely connected to the Charger terminals.
- c. The following lights on the Charger Unit indicate the battery charge status:

<p>One red light.</p> 	<p>Power on. The Charger Unit is receiving power. No battery is connected.</p>
<p>Two red lights.</p> 	<p>Full charge mode. The battery is low and charging.</p>
<p>One red light, one green light.</p> 	<p>Trickle charge mode. The battery is 90% charged. Leave the battery on the charger for another one to two hours to reach 100% charge.</p>

If a battery is left in the charger for a significant period of time after it is fully charged, it will automatically stop charging and display a flashing red light.

<p>Two red lights, (one flashing).</p> 	<p>Charging stopped. Remove from charger.</p>
---	--

2. Attach the lamp head to loupes using the supplied mount.

- a. **For Miltex®, PeriOptix®, Surgitel®, Orascoptic®, Zeiss® GF and other flip style loupes:**
Slide clip onto front of interpupillary cross bar of loupes, with one circular cut out on top and one cut out on the bottom of the cross bar. The clip should securely snap into place.
- b. **For through-the-lens (TTL) solid bridge mount:** Loosen the thumb nut on bridge clamp, opening the two halves wide enough to engage the top and bottom of frame nose bridge. Attach the clamp to the bridge and tighten nut. Note: the tightening nut should be oriented up (towards the ceiling) when the clamp is attached to the nose bridge.

3. Remove the battery from the charger unit and slide into the battery case.
The battery should snap into place.

4. Clip the battery case to your waist band or pocket.

- a. Attach the lamp cord to the loupes frames with the small elastic strap.
- b. Use the shirt clip on the lamp cord to position the cord on your shirt or lab coat as desired.

5. Turn on lamp using on/off switch.

- a. The intensity control knob will adjust lamp intensity.

6. Adjust the direction of the LED beam.

- a. To adjust the tension of the lamp head position, use the supplied Philips screwdriver to tighten or loosen the two (2) small link screws as needed.

For any questions or problems, please call Miltex Customer Service at 866-854-8300

Warranty: Miltex warrants its products to be free of original defects in material and workmanship and to perform in accordance with specifications for the following terms:

30 Day Trial Period:

LED lamp head, cables, chargers and battery pack. If any issues should arise during the first 30 days, Miltex will replace the above components free of charge.

Limitations of Warranty:

- Breakage or failure due to tampering, misuse, neglect, accidents, modification or shipping.
- If the product is not used in accordance with manufacturer's recommendations or instructions.
- If repaired or serviced by other than Miltex or a Miltex authorized representative.
- Prompt inspection and reporting of missing or damaged product should be made to shipping carrier and Miltex with 3 days of receipt of product.
- If Miltex products or any component thereof are found to be defective or at variance with the manufacturer's specifications during the warranty period, Miltex will repair or replace the product or component(s) at no cost to the purchaser. This warranty only applies to products purchased new from Miltex or its authorized distributors or representatives.
- The purchaser must return the product directly to Miltex or an authorized distributor or representative and bear the costs of shipping.
- This is the only warranty made by Miltex. Miltex makes no other warranty, express or implied, including any warranty of merchantability and fitness for a particular purpose.

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Caution

Consult Instructions for Use

Product complies with requirements of directive 93/42/EEC for medical devices

Batch Code or Lot Number

Catalog Number

Manufacturer

Authorized Representative in the European Community

Latex-free

Rx ONLY Rx Only: Caution: Federal (US) law restricts this device to sale by or on the order of a physician.

Integra York PA, Inc.
589 Davies Drive
York, PA 17402 USA
toll free phone: 866-854-8300
phone: 717-840-9335
fax: 717-840-9347
www.miltex.com

Integra LifeSciences Services (France)
SAS Immeuble Sequoia 2
97 allée Alexandre Borodine
Parc Technologique de la Porte des Alpes
69800 Saint Priest-France
Telephone: 33 (0) 4 37 47 59 10
Fax: 33 (0) 4 37 47 59 29

Miltex and Miltex Redefining Excellence are registered trademarks of Miltex, Inc., a subsidiary of Integra LifeSciences Corporation.
The Integra wave logo is a trademark of Integra LifeSciences Corporation.
All other trademarks are property of their respective owners.
Copyright© 2010 Miltex, Inc.

LPSLGHTDFU

Rev. B 08/13

+H834LPSLGHTDFU0U