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**INTEGRATOR**  
**CHIROPRACTIC ADJUSTING INSTRUMENT**  
**U.S PATENT NO. 5632765, Foreign Patents Pending**

**INDICATION:**  
For the correction of Vertebral Subluxation

**DIRECTIONS:**  
Apply integrator to the vertebral subluxation to be treated

**CAUTION:**  
Federal law restricts this device to sale by, or on the order of, a licensed practitioner

**CARE INSTRUCTIONS:**

- Do not sterilize. May damage parts. (The removable pisiform tips may be cleaned between use by dry wipe or with an alcohol wipe.)
- Integrator may be wiped clean using a surface disinfectant or dilute alcohol solution, Do not submerge or saturate with water or solvents.
- Integrator tip may be treated with a surface disinfectant spray between patients.
- No lubrication is required. Do not disassemble.
- No special care required for storing.
- Premature failure will occur if Mallet is activated without surface contact

**CONTRAINDICATIONS**

**DO NOT USE ON:**

- Fractures
- Soft tissue including, not limited to:
- Eyeballs, ovaries, testicles
- Mammary glands
- Auditory canals

Should your Integrator require service, please contact the Returns Department to obtain a Return Material Authorization number and instruction to return the instrument.

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BREVETTO U.S.A. N. 5632765, brevetti di altri Paesi in corso di approvazione