NASAL OR SINUS DRESSINGS

INDICATIONS

- Post operative packings
- Epistaxis

PRECAUTIONS:

In rare instances, the physiochemical conditions associated with nasal and sinus surgery, both with and without nasal and sinus packings, may present a risk of toxic shock syndrome (TSS). Warning signs of TSS include sudden fever (usually 102° or more), vomiting, diarrhea, dizziness, fainting (or near fainting when standing up), and/or a rash that looks like a sunburn. Integra Miltex nasal and sinus packings exhibit no antimicrobial properties, they are not bacteriostatic towards pre-existing infections, nor do they prevent the occurrence of new infections. In the case of pre-existing infections, it is recommended that appropriate treatment be instituted.

- Always moisten before removal
- Do not leave packing in for more than four days
- This product must be disposed of per standard protocol for biohazardous materials due to potential exposure to blood borne pathogens

BEFORE INSERTION

- Trim dressing and/or airway tube if required
- Apply antibiotic ointment over the surface

TO INSERT

- Grasp packing with forceps or fingers
- Gently advance posteriorly along the nasal floor until the entire packing is within the nose

AFTER INSERTION

- Have the patient tilt head back and irrigate end of packing with 5-10cc of saline or antibiotic solution contained in a syringe to fully expand packing
- For epistaxis - saline solution may not be required

BEFORE REMOVAL

- All nasal and sinus dressings MUST be irrigated with either saline or water (minimum 10cc) prior to removal
- With packings containing airways, first remove airway tube and then irrigate into space previously occupied by the tube

TO REMOVE

- Use forceps to grasp the pack and gently remove

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SYMBOLS

	Caution
	Consult Instructions for Use
	Manufacturer
	Authorized Representative in the European Community
	Product complies with requirements of directive 93/42/EEC for medical devices
	Caution: Federal (USA) law restricts this device to sale by or on the order of a dentist or physician.
	Catalog Number
	Lot Number
	Do Not Reuse
	Quantity
	YYYY-MM
	Sterilized using irradiation

Integra York PA, Inc.
 589 Davies Drive
 York, PA 17402 USA
 toll free phone: 866-854-8300
 phone: 717-840-9335
 fax: 717-840-9347
 www.integrallife.com/integra-miltox

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