[REF] 75-20 Screw type **[REF] 75-22** Snap on

[REF] 75-20 Tipo atornillado **[REF] 75-22** A presión

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Precautions

- The contra angle must be lubricated before every autoclaving procedure.
- Damage due to lack of lubrication is misuse and is not covered by the warranty.
- Maximum autoclave temperature is 135°C (275°F).
- Do not use dry heat or cold sterilization.

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Lubrication Instructions

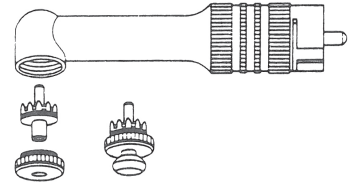
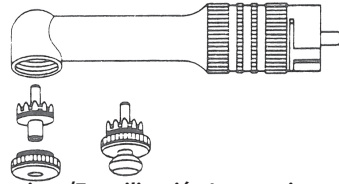
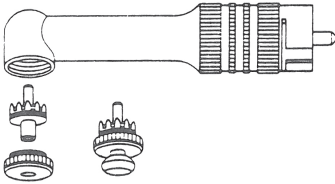
1. Remove cup or brush.
2. Thoroughly clean the outside of the angle with a disinfectant moistened gauze sponge.
3. Unscrew the knurled head cap by turning clockwise and remove the cap and screw stem holders. The snap on style angles have a single unit cap, ball and gear assembly.
4. Lubricate all gears with a hand piece lubricant and reassemble, making sure that the knurled cap is tightened firmly counterclockwise.
5. Rotate the shaft several times to circulate the lubricant and to make certain that the gears have intermeshed properly.
6. Wipe off any excess lubricant expelled.
7. Sterilize by autoclaving before putting back into service.

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Cleaning/Sterilization Instructions

1. Thoroughly clean the outside of the angle with a disinfectant moistened gauze sponge.
2. Autoclave angle per manufacturer's instructions. The contra angles have been tested with no adverse effects at 130-135° C (266-275°F) with a 20 minute exposure time.

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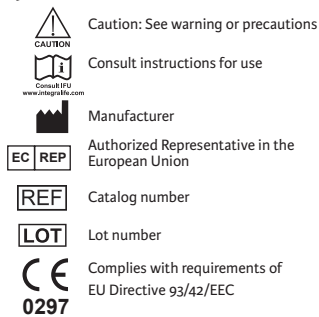
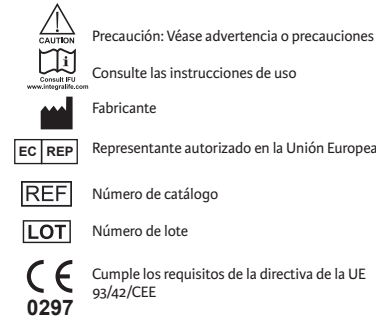
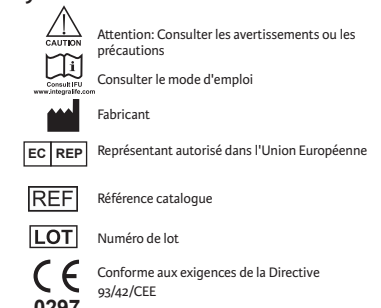
Símbolos

Symboles

Rx ONLY Caution: Federal (USA) law restricts this device to sale by or on the order of a dentist or physician.

Integra York PA, Inc. • 589 Davies Drive, York, PA 17402 USA
866-854-8300 USA • 717-840-2763 outside USA • 717-840-9347 fax
integralife.com/integra-miltex

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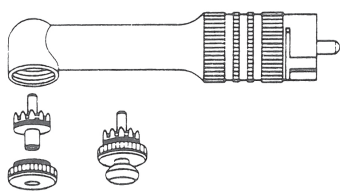
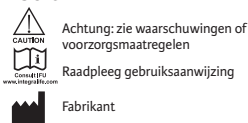
Integra and the Integra logo are trademarks of, and MilteX is a registered trademark of, Integra LifeSciences Corporation or its subsidiaries.
 © 2011 Integra LifeSciences Corporation. All rights reserved.

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[EC REP] Integra LifeSciences Services (France) SAS Immeuble Sequoia 2